



# iRelaxBot

## OPERATIONAL FLOW CHART



### PRE-USE INSTRUCTIONS

iRelaxBot IS NOT BE ALLOWED TO USE THE DEVICE IF

1. Patients with severe heart disease (including those equipped with pacemakers or using other medical electronic devices).
2. Patients with severe malignant tumors.
3. Delirium and psychosis.
4. Severe cases of bone tuberculosis, osteoporosis, bone injury and spinal deformity.
5. Female menstrual period, pregnant or shortly after.
6. Patients with cerebral hemorrhage and fever within one year after abdominal surgery.
7. Do not use this machine when the body is wet.
8. Patients with space-occupying lesions, infectious diseases and severe essential hypertension in the spinal cord and brain.
9. Consciousness disturbance within half an hour after meal.
10. Children and drunkards are not allowed to use this machine.
11. It is not suitable for patients with local skin damage, ulcer and bleeding.
12. It should not be used within half a year after the operation of the fracture or by people who have not completely healed after the operation.



### PRE-USE PREPARATION

IF YOU ARE WEARING ANY OF THE FOLLOWING ITEMS, PLEASE REMOVE THEM:



GLASSES



EARRINGS, STUDS



MOBILE PHONE



THE KEY



### USING 3 STEPS

#### The 1st step

Take off the shoes, take on the shoe cover, put the feet into the foot air bag, foot bottom should touch the bottom air bag;



#### The 2nd step


Lie in the middle of the bed, the spine is right on the back air bag, neck pillow on the head feng fu point position;



#### The 3rd step

Select physical therapy mode on the hand controller, close your eyes, relax your body and start to enjoy physical therapy.

720 ° REGULATE	360 ° REGULATE	ACOUSTIC VIBRATION	T-DEGREE PRESSURE	OVERSPEED SIEVE	SELF-WEIGHT TRACTION
30MIN	15MIN	10MIN	10MIN	10MIN	10MIN

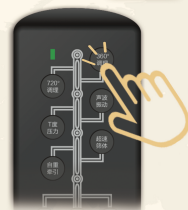

iRelaxBot